ACCU-TRAC®

Precision Die System

Instructions for Use

Intended Purpose

ACCU-TRAC Precision Die System is intended to act as a base and retention system for models and dies for use in the creation of dental restorations. These models and dies can be used as tools for planning for the appropriate restoration such as crowns, bridges, implants, combinations, or other dental prostheses.

Intended User

The intended user population are dentists, hygienists, and laboratory technicians in a dentist office. The ACCU-TRAC Precision Die System is not intended to be used by the general population.

Intended Patient Population

The ACCU-TRAC Precision Die System is not used on patients or by patients. It is used by dentists, hygienists, and laboratory technicians in a dental office.

Clinical Benefits

The ACCU-TRAC System provides the clinical benefit of effective fabrication of dental models and dies for the creation of dental restorations.

Contraindications

The ACCU-TRAC System does not have any contra-indications.

Residual Risks

Failure to follow instructions provided in this IFU may result in inaccurate or incomplete model fabrication.

Avoid using excess model material to ensure proper function of the magnetic retention system.

Use caution when operating locking arms to avoid breakage, damage to the model, pinching, or other hand injury.

Avoid using this device around magnets as this may interfere with the magnetic retention system and may damage the model.

Avoid using this device around electronic devices as the magnets may cause electromagnetic interference.

Symbols Glossary

Symbol	Meaning
	Consult the instructions for use
	Medical Device Indicator: this product meets the definition of a medical device
	Sale Restricted: the purchase of the device is restricted to healthcare practitioners only
	Date of manufacture
	COLTENE catalog number
	CE Mark
	Indicates the entity importing the medical device
	The entity that manufactures the medical device
	Batch code
	Authorized Representative in the European Community

Notice: Report any serious incident that has occurred in relation to this device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Upon request, a hardcopy of the IFU will be supplied free of charge.

ORDERING INFORMATION

TYPE	ITEM	CAT NO.	DESCRIPTION
Standard Packaging	Introductory Kit	AT6	6 Accu-Trac Precision Die Trays 6 Pouring/Ejector Bases 25 Articulation Keepers 25 Return Request Cards 25 Patient Identification Labels
	Refill Kit	ATR12	12 Accu-Trac Precision Die Trays 50 Articulation Keepers 50 Return Request Cards 50 Patient Identification Labels
Economy Packaging	Introductory Kit	AT72	12 AT6 Kits, 72 Units
	Refill Kit	ATR144	12 ATR12 Kits, 144 Units
Accessories	Articulation Keepers	ATK300	300 Articulation Keepers 300 Return Request Cards 300 Patient Identification Labels
		ATK25	25 Articulation Keepers
	Locking Arms	ATA20	20 Sets Locking Arms
	Model Ejectors	ATE12	12 Pouring/Ejector Bases
	Articulation Spacers	ATS20	20 Articulation Spacers
	Accu-Trans®	ATN50	50 Disposable Carrying Cases
	Transport System		50 Plastic Retainers 50 Doctor/Patient Labels

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

ACCU-TRAC®

Precision Die System

MODEL FABRICATION

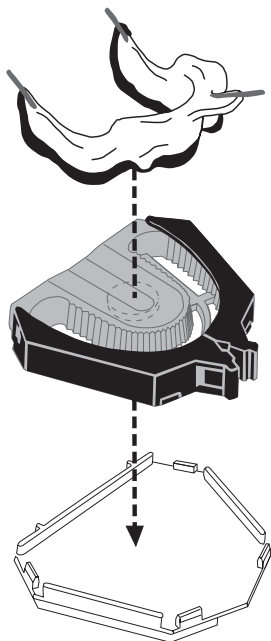
Fig. 1

1. Assemble Accu-Trac with arms locked and base attached.
2. Reduce height of model, by eliminating excess flange of tray and impression material.
3. Create three reference marks on the impression tray for lateral and anterior posterior alignment. Place one reference mark at the anterior midline and the other two at the center of the posterior borders.
4. Pour impression. Pour remaining stone in Accu-Trac, level with the top of locking arms. Invert poured impression and place on Accu-Trac.
5. Align the 3 reference marks of the impression tray to the midline and posterior reference marks of Accu-Trac (Fig. 1).
6. Remove excess stone from edge of arms and palatal area prior to final set.

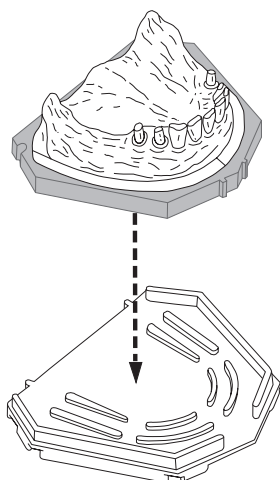
Fig. 2

7. Remove impression from model after stone has set. Remove base and arms from Accu-Trac.
8. Reverse base into eject position and eject model by carefully pressing Accu-Trac toward base with uniform pressure to avoid model breakage (Fig. 2).
9. Trim peripheral edges of model as needed. Avoid trimming die contact tracks.
10. Saw any remaining stone from palatal or lingual area and section dies in parallel alignment with tracks.
11. Wash clean, and dry dies carefully.
12. Reassemble dies to Accu-Trac in numerical order and secure sectioned model by locking arms into position.

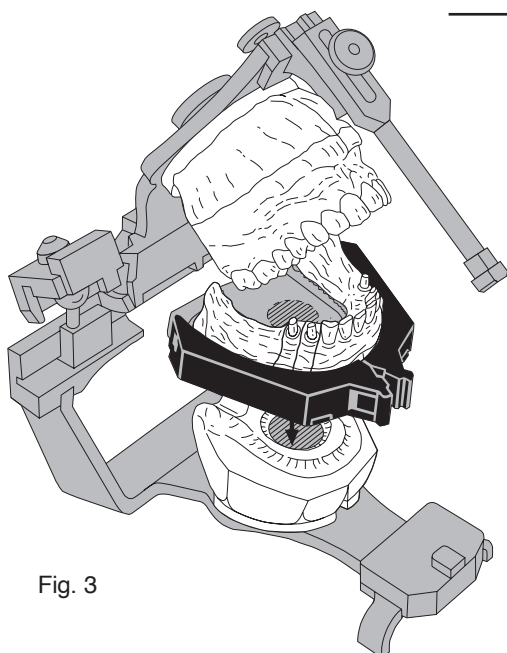
Fig. 3

ARTICULATION

1. Place articulation Keeper on Accu-Trac magnet.
2. Snap prefabricated green Spacer over Keeper onto Accu-Trac tray.
3. Cover Keeper and prefabricated Spacer with plaster. Articulate model conventionally.
4. Remove model from articulator. Peel away prefabricated Spacer (Fig. 3).

***NOTE:** Also available, ACCU-TRANS® Transport System (ATN50), disposable carrying case for the model and finished case.

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